

# HS-WC-003 Defibrillator Wall Cabinet

## with alarm and red flashing light

Cabinet approximate external dimensions:  
Width 355mm Height 460mm Depth 145mm

### Installation Guide and Operating Instructions

1. Ensure that there are no electrical or other services in the wall where the cabinet is to be mounted
2. Open the cabinet door and alarm & light will sound.
3. Locate the door plunger switch in the top left corner and grasp the plunger with your fingers.
4. Pull the plunger out towards the front of the cabinet. This places the switch into maintenance position (override) for easy cabinet installation
5. Mount the wall bracket at a suitable height using appropriate fixing methods
6. Hang the cabinet on the wall bracket. The cabinet will be removable by lifting it off the bracket. (By following steps 4 and 5 you will be able to mount it so that the cabinet is non-removable)
7. Mark the positions of the 2 lower fixing holes, visible from the inside of the cabinet
8. Secure the cabinet using appropriate fixing methods through these holes
9. After removing the HeartSine Samaritan defibrillator from its box and following the instructions provided in the operating manual, hang the defibrillator on the hook inside the cabinet by the carry strap.
10. Close the cabinet door and latch it. The switch will automatically reset to operational position.
11. Check operation by opening the door and closing it again.
12. The cabinet installation is now complete.



### Connection to Existing Alarm Systems

1. On request, we can configure the cabinet to connect to an existing alarm system. Customers however, will have to allow one week production time.
2. The installation and connection to the alarm system is organised by the customer. Aero Healthcare provides the configured cabinet which can be connected to the circuitry for the customer to install.

### Troubleshooting:

- Battery replacement should only be necessary after several years. A simple monthly check is to briefly open and close the cabinet door to ensure that the alarm and light are functioning properly. When the battery is low the alarm might function without the light. To replace the battery, release the battery holder from beside the switch, replace battery and clip back into place.